

# **INSTRUCTIONAL GUIDE: REUSABLE FACE MASK**

Ashley Harris


Product Engineer - Trim

# FACE MASK COVER INSTRUCTION - Materials



## MATERIALS:

- Thin width elastic 16" long
- Main Fabric 12" X 6" prewashed
- Lining Fabric 10" X 6" prewashed
- Face Mask Sewing Pattern (print out and check scale to verify)
- Thread
- Ruler
- Sewing Machine, but can be hand sewn

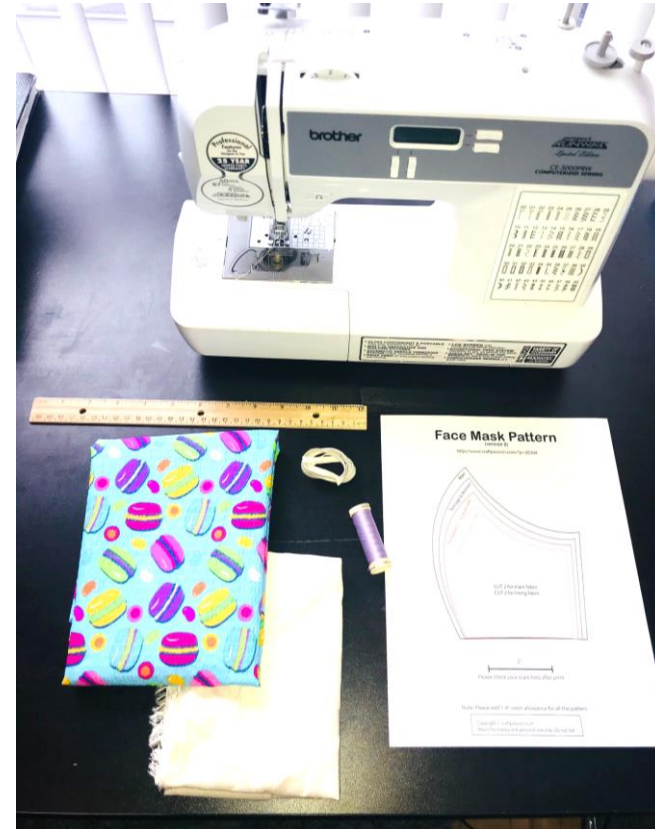

# FACE MASK COVER INSTRUCTION – Print Pattern




- Print out facemask sewing pattern
  - <https://www.craftpassion.com/wp-content/uploads/2013/08/face-mask-pattern-1.pdf>
  - Do not scale, it is created for paper size 8.5" X 11"
- Verify size using 2" scale on pattern
- Determine and cut out size needed from paper pattern

## Face Mask Pattern

(version 2)

<http://www.craftpassion.com/?p=26304>

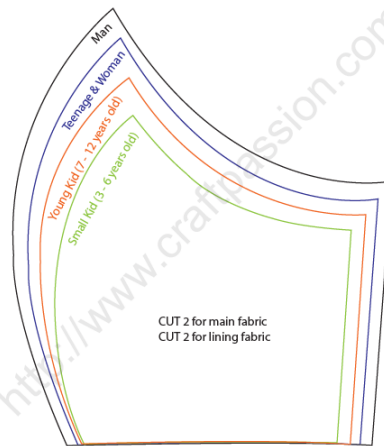


2"

Please check your scale here after print

Note: Please add 1/4" seam allowance for all the pattern


Copyright © craftpassion.com  
Meant for hobby and personal use only. Do not sell.

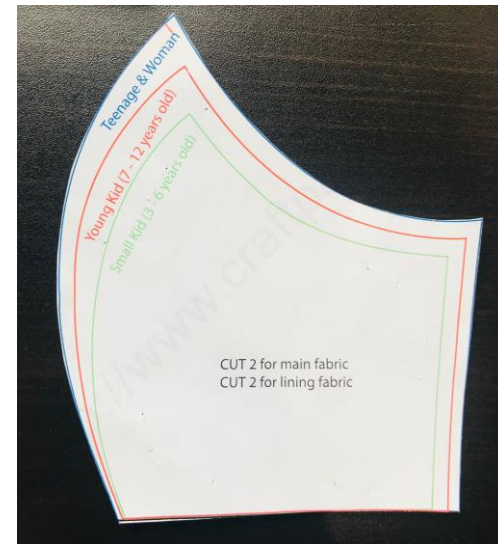

# FACE MASK COVER INSTRUCTION

## Add Seam Allowance and Cut Outer Pieces


- Fold main fabric in half
- Pin paper pattern to folded fabric
- Mark up a  $\frac{1}{4}$ " seam allowance at top, curved edge and bottom of pattern
- Mark up a 1" seam allowance at ear side
- Cut out pieces while stacked and pinned
- You should now have 2 outer pieces

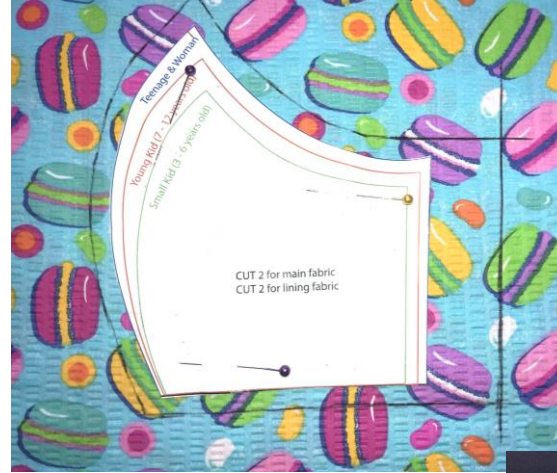

# FACE MASK COVER INSTRUCTION

## Add Seam Allowance and Cut Lining Pieces

- Fold lining fabric in half
- Pin paper pattern to folded fabric
- Mark up a  $\frac{1}{4}$ " seam allowance at top, curved edge and bottom of pattern
- Do not include a seam allowance at the ear side on the lining piece
- Cut out lining pieces
- You should now have 2 lining pieces

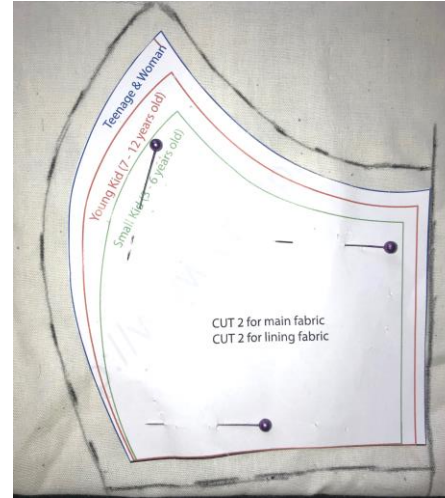

# FACE MASK COVER INSTRUCTION – Sewing

- Place outer pieces together with “right” sides of fabric facing each other
- Sew centerline of face mask
- Fold selvedge to one side and top stitch down about 3mm from join seam





# FACE MASK COVER INSTRUCTION – Sewing

- Repeat this step for the lining pieces



# FACE MASK COVER INSTRUCTION – Sewing


- Place Lining inside out on top of outer piece while outer piece is sitting right side up
- Align center seams and pin together
- Sew top and bottom lines at  $\frac{1}{4}$ " seam allowance





# FACE MASK COVER INSTRUCTION – Sewing

- Turn face mask right side out
- Press seam and pin
- Top stitch about 3mm from edge

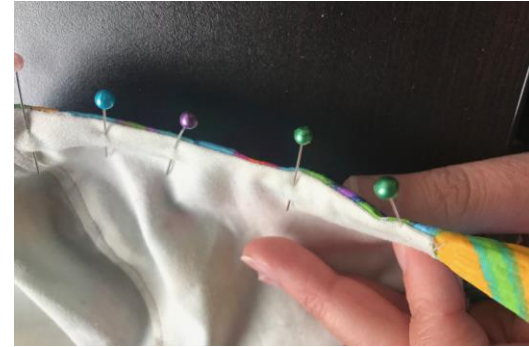

# FACE MASK COVER INSTRUCTION

## Create Elastic Band Pocket

- Outer fabric should be 1" longer than lining fabric
- Lay mask outer piece facing down
- Fold raw edge of outer piece in  $\frac{1}{4}$ " and press with iron
- Fold again and align  $\frac{1}{4}$ " past raw edge of lining
- Sew a straight line down
- Repeat on both sides



# FACE MASK COVER INSTRUCTION

## Add Elastic Ear Bands

- Cut 2 pieces of elastic 8" long – try to use a thin piece of elastic either a cord or up to about 3/8" wide
- Insert elastic through casing that you just created
  - If it is difficult to get elastic through casing, connect a safety pin to one end of the elastic and make sure its closed. Then guide elastic through casing using the safety pin. Disconnect after completing this step





# FACE MASK COVER INSTRUCTION

## Add Elastic Ear Bands

- Place and pin both ends of elastic together and sew – you can also tie them together if your elastic is too small to sew
- Pull elastic so that your sewn or tied edge is inside the casing
- Repeat on the same steps on the other side



# FACE MASK COVER INSTRUCTION

## Completed Mask Cover



FRONT VIEW



INSIDE



*Disclaimer: Magna makes no warranties of any kind and does not guarantee any quantity, quality, or compliance to any standards of these masks. Magna assumes no liability for any damages or loss of any kind that might arise from the use of, misuse of, or the inability to use the masks.*



DRIVING **EXCELLENCE.**  
INSPIRING **INNOVATION.**